

**ULTRA
MED**



شركة القيرا
للمنتجات الطبية

Ultra for medical Products (ULTRAMED) U.M.I.C (S.A.E)

INSTRUCTION FOR USE

Material Used:

- Polyethylene.
- Sponge
- Medical Grade adhesive.

Indications:

- To collect the urine for both male and female infants.

Contraindications:

- Can cause infection to the patient.
- Not to be used in patient allergic to adhesive

Instruction for Use:

FEMALE PATIENTS

- First clean and dry the perineal area thoroughly removing any natural or applied skin oils. Remove the protective backing sheet to expose the adhesive area and opening.
- Apply the bag by first placing the opening directly in front of the rectum.
- Excess of bag should be towards feet.
- Spread the labia and gently press the adhesive area down: then release the labia.
- Complete the application by pressing the adhesive base area around the upper genitalia. The bag is now in position and ready for the short-term specimen collection.
- The adhesive is non-sensitizing and will remove comfortably.
- Reapply diaper, if required, directly over bag.
- It must be used by skilled and trained personnel.



MALE PATIENTS

- Thoroughly clean and dry the scrotal area to remove any natural or applied skin oils.
- Remove the protective backing sheet to expose adhesive area and opening.
- Carefully place the penis well into the bag.
- Excess bag should be towards feet.
- Press adhesive base firmly against scrotal area.
- The bag is now in position and ready for the short-term specimen collection.
- The adhesive is non-sensitizing and will remove comfortably.
- Reapply diaper, if required, directly over bag.
- It must be used by skilled and trained personnel.

Instruction For Use

ISSUE/REV: - 2/0

Page 1 of 2

PEDIATRIC URINE BAG

**ULTRA
MED****شركة الترا
للمنتجات الطبية****Ultra for medical Products (ULTRAMED) U.M.I.C (S.A.E)****AFTER REMOVAL**

- The top of the collector is sealed securely by folding and sealing the adhesive area. This will help prevent spillage and maintain the cleanliness of the specimen.

Cautions:

- Do not use if package is open or damaged.
- Discard after single use.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room Temperature

EC REP**Obelis^{SA}**

53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net

CE
2803

Sterilized by Ethylene Oxide



For Single Use Only



Cautions

LOT

Lot No.



Date of Manufacturing



Use by / Expiry date

MD

Medical Device



Non-Pyrogenic

REF

Catalogue No.



Manufacturer Address

phthalate



Latex Free



Don't use if package is not intact



Consult Instruction for Use

**ISSUE/REV DATE:** 01.09.2022**Instruction For Use**

ISSUE/REV: - 2/0
Page 2 of 2

PEDIATRIC URINE BAG