



ULTRA FOR MEDICAL PRODUCTS CO.

INSTRUCTION FOR USE FOR NELATON CATHETER

Material Used:

- Medical grade phthalate free P.V.C. (Poly Vinyl Chloride) (DEHP Free).
- Colored master batch.

Indications:

- To drain the bladder in case of urinary retention.
- Collection of sterile urine sample.
- Provide relief of discomfort from bladder distention.
- Decompression of the bladder.
- Measure residual urine.
- Management of patients with spinal cord injury, neuromuscular degeneration, or incompetent bladders.

Contraindications:

- Unexplained haematuria
- Previous lower abdominal surgery
- Blood clotting disorders
- Ascites
- Suspicion of ovarian cyst

Instruction for Use:

- Clean the genitalia of the patient with mild antiseptic solution.
- Put some antiseptic jelly on the tip of the catheter without touching the tip of the catheter.
- Advance the catheter into the urethra to its full length to the bladder.
- Once the catheter reaches the bladder the urine starts to come out. Collect the urine in to the urine bag.

Cautions:

- Do not use if package is open or damaged.
- Discard after single use.

Warning (consequences of re-use of the device):

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.
-

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of direct Sun light.
- Room temperature.



ULTRA FOR MEDICAL PRODUCTS CO.



Obelis^{SA}

53 Bd. Général Wahis
B-1030 Brussels, Belgium

Phone: 32.2.732.59.54

Fax: 32.2.732.60.03

E-mail: mail@obelis.net

www.obelis.net



Sterilized by Ethylene Oxide



For Single Use Only



Cautions



Lot No.



Date of Manufacturing



Use by / Expiry date



Medical Device



Manufacturer Address



Non-Pyrogenic



Don't use if package is not intact



Non-phthalate



Latex Free



Consult Instruction for Use

ISSUE/REV DATE: 01.09.2022

Assiut Factory: Part No. (304:310) & part no. (312) – Industrial Area, Arab El Awamer – Abnoub - Assiut – Egypt. Tel: 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax: 002-088-4964222 **& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaay Triumph Square Heliopolis Tel: 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

Assuit Office: 23, July Str. Tel: 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

Alexandria Office: 212Abd El salaam Airef Str, Luran Tel: 03/5856202– 5856458 **& Fax:** 03/58289888 **& Mob:** 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com

Instruction For Use

ISSUE/REV: - 2/0

Page 2 of 2

STERILE NELATON CATHETER