

INSTRUCTION FOR USE FOR MUCUS EXTRACTOR

Material Used:

- Medical grade phthalate free P.V.C. (Poly Vinyl Chloride) (DEHP free).
- Poly Carbonate.
- H.D.P.E. (High Density Polyethylene)
- Master Batch (Green)

Indications:

- To clear the secretion from the oral cavity.

Contraindications:

- Can damage the soft tissue of nasal route.
- It may cause sinusitis, esophagitis, gastro-ulceration and pulmonary infections.

Instruction for Use:

- Connect the funnel (connector) the suction machine.
- Put the plain tube into the mouth patient and suck the secretion from the funnel side.
- The secretion is collected into the container.
- Put the plain tubing into the nostril of the patient and suck the secretion.
- Remove the tubing from the patient. Remove the lid of container.
- Fix the other lid supplied with the packing & send the container for the laboratory testing.

Cautions:

- Do not use if package is open or damaged.
- Discard after single use.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature.



Obelis^{SA}
53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide



For Single Use Only



Cautions



Lot No.



Date of Manufacturing



Use by / Expiry date



Medical Device



Manufacturer Address



Latex Free



Non-Pyrogenic



Don't use if package is not intact



Consult Instruction for Use



Catalogue No.

Asheer



ISSUE/REV DATE: 01.09.2022

Assiut Factory: Part No. (304:310) & part no. (312) – Industrial Area, Arab El Awamer – Abnoub - Assiut – Egypt. Tel: 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax: 002-088-4964222 & **Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaay Triumph Square Heliopolis Tel: 022/4171621-4143794 & Fax: 022/4171613 & Mob: 01223988200

Assiut Office: 23, July Str. Tel: 088/2364111 – 2364222 & Fax: 088/2334964 & Mob. : 01223988202

Alexandria Office: 212Abd El salaam Airef Str, Luran Tel: 03/5856202– 5856458 & Fax: 03/5828988 & Mob: 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com