

**ULTRA
MED****شركة القترا
للمنتجات الطبية****Ultra for medical Products (ULTRAMED) U.M.I.C (S.A.E)**

INSTRUCTION FOR USE FOR URINE COLLECTION BAG LEG BAG

Material Used:

- L.D.P.E. (Low Density Polyethylene)
- Master Batch (Colored).
- P.P. (Polypropylene)
- H.D.P.E. (High Density Polyethylene)
- P.V.C. (Poly Vinyl Chloride) non phthalate and DEHP free.
- P.V.C. (Poly Vinyl Chloride) sheets.

Indications:

- Used to collect the urine of the patient from the bladder in case of urinary retention through Nelaton or Foley's catheter and to measure the volume of urine for daily checks.

Contraindications:

- Can cause infection to the patient.

Instruction for Use:

- Strap the bag to the leg of the patient.
- Remove the cover of inlet connector from the tube and connect the tube with the Foley's or Nelaton catheter.
- The measuring scale on the bag provides an approximate and indicative measurement of the volume of the urine.
- Drain the urine using outlet connector if the bag is filled more than half of the capacity.
- It must be used by skilled and trained personnel.

Cautions:

- Discard after single use.
- Use only if the package is intact

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
 - Room temperature.

**Obelis^{SA}**

53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net

**Non-Sterile****For Single Use Only****Latex Free****See Instruction for use****Medical Device****Lot No.****Date of Manufacturing****Use by / Expiry date****Catalogue No.****Manufacturer Address****Don't use if package is not intact****ISSUE/REV DATE: 01.09.2022****Consult Instruction for Use**

Signature
Asther

Instruction For Use

ISSUE/REV: - 2/0
Page 1 of 1

URINE BAG LEG BAG