



ULTRA FOR MEDICAL PRODUCTS CO.

INSTRUCTION FOR USE

Material Used:

- P.P. (Polypropylene)
- A.B.S. (Acrylonitrile butadiene styrene)
- Medical grade phthalate free P.V.C. (Poly Vinyl Chloride).
- Master Batch (Blue)
- Master Batch (White)
- H.D.P.E. (High Density Polyethylene)
- Latex Bulb.
- Stainless Steel

Indications:

- The Hemo Set is used to administer Blood into human circulating system by using intravenous catheter or Cannula. The Hemo Sets are specially designed to administer measured volume of blood through gravity.

Contraindications:

- Not to be used in patients with known hypersensitivity to any of the materials used.

Instruction for Use:

- Close all the controllers.
- Remove the spike cover and insert the spike firmly into blood container.
- Suspend the solution container with the attached set and open the upper chamber. Close the upper clamp.
- Gently squeeze and release the drip chamber until it is one third full.
- Remove the needle cover & slightly open lower clamp to remove air from the tubing and vein needle. Close the lower clamp.
- Open the upper clamp & allow blood to flow into graduated chamber until the desired volume is obtained. Do not wet air filter by too much blood. Close the upper clamp.
- Make vein puncture.
- Gradually open the lower clamp. Adjust drip rate & control blood with the clamp. The valve will shut off the flow when solution level in graduated chamber is zero.
- When more blood is required, close the lower clamp & open the upper clamp to fill graduated chamber until desired level. Close the upper clamp and squeeze the drip chamber a little to float the valve up and restart infusion.

Warning:

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

Cautions:

- Do not use if package is open or damaged.
- Use rubber part only for injecting the medicines.
- Do not use if protective caps loose or missing
- Discard after single use.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature .

Instruction For Use

ISSUE/REV: - 2/0

Page 1 of 2

Sterile Hemo SET



ULTRA FOR MEDICAL PRODUCTS CO.

ULTRA FOR MEDICAL PRODUCTS CO. DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.



Obelis^{SA}
53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide



Lot No.



For Single Use Only



Date of Manufacturing



See Instruction for use



Use by / Expiry date



Catalogue No.



Non-Pyrogenic



Don't use if package is not intact



Non-phthalate

ISSUE/REV DATE: 20 -12- 2020



Assiut Factory: From Part No. 304 to Part No. 310 and Part No. 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax:002-088-4964222**& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaay Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

Assiut Office: 23, July Str. Tel: 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

Alexandria Office: 212Abd El salaam Aref Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988**& Mob:** 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com

Instruction For Use

ISSUE/REV: - 2/0

Page 2 of 2

Sterile Hemo SET