

INSTRUCTION FOR USE

Material Used:

- P.P. (Polypropylene)
- Medical grade phthalate free P.V.C. (Poly Vinyl Chloride) (DEHP free).
- H.D.P.E. (High Density Polyethylene)
- L.D.P.E. (Low Density Polyethylene)
- Stainless Steel

Indications:

- Drainage or collection of fluid from surgical wound.

Contraindications:

- Not to be used in patients with known hypersensitivity to any of the materials used.

Instruction for Use:

ULTRAVAC SET

CLOSED WOUND SUCTION UNIT

ULTRAVAC SET has been specially designed to offer the Medical Fraternity an effective device for close wound drainage under negative pressure with the option to use one catheter or two catheters simultaneously.

UNIT CONSISTS OF :

1. Bellow unit with connector and hanger.
2. Connecting tube with clamp and "Y" connector.
3. Curved needle with matching catheter.
4. Spare perforated catheter.

(Catheter sizes: FG 10 , 12 , 14 , 16 , 18)

INSTRUCTION FOR USE :

1. (a) Place the perforated tube in the wound with the help of curved needle , as (Fig 1&2) Ensure that the entair perforated part of the catheter is well within the wound to avoid any air leakage during drainage.

- (b) The length of the perforation should be reduced according to the size of the wound.
- (c) Catheter is marked 5 cm from the last perforation to ascertain the depth of catheter in the wound.
- 2- Press the bellow container with both hands.(Fig. 3)
- 3- Connect the connecting tube (duly clamped off) to the bellow container. (Fig. 4)
- 4- Connect the perforated catheter with connecting tube (Fig. 5) and start drainage by releasing the clamp.
- 5- To re-charge the vaccum, start operation repeating instructions (2,3,4).

FOR USE OF TWO CATHETERS SIMULTANEOUSLY

Remove protector cap provided at the "Y" connector and connect the spare perforated catheter with the "Y"connector as shown in (Fig. 6) and follow the rest of the directions as given above.

STERILE EO

EC REP
Obelis SA
Bd.Général Wakis, 53
B-1030 Brussels,Belgium
www.obelis.net

ULTRA MED شركة التـسـرا
للمنتجات الطبية
(U.M.I.C)

MD **CE**
PYROGEN FREE 2803

Manufactured by Ultra for medical Products (ULTRAMED) U.M.I.C (S.A.E)
Factory: From part no. 304 to part no. 310 and part no. 312
Arab El Awamer - Industrial zone - Abnoub - Assiut, Egypt
www.ultramedumic.com
صنـع في مـصر NV.0822



ULTRA FOR MEDICAL PRODUCTS CO.

Warning:

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

Cautions:

- Do not use if package is open or damaged.
- Discard after single use.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
 - Nice Ventilated place.
 - Out of Sun light.
- Suitable temperature (18 – 32 °C).



Obelis^{SA}

53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide



For Single Use Only



Cautions



Lot No.



Date of Manufacturing



Use by / Expiry date



Medical Device



Non-Pyrogenic



Catalogue No.



Manufacturer Address



Don't use if package is not intact



Non-phthalate



Latex Free



Consult Instruction for Use

ISSUE/REV DATE: 01.09.2022

Signature
Asther

Assiut Factory: Part No. (304:310) & part no. (312) – Industrial Area, Arab El Awamer – Abnoub – Assiut – Egypt. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax: 002-088-4964222 & **Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 & **Fax:** 022/4171613 & **Mob:** 01223988200

Assiut Office: 23, July Str. **Tel:** 088/2364111 – 2364222 & **Fax:** 088/2334964 & **Mob. :** 01223988202

Alexandria Office: 212Abd El salaam Aired Str, Luran **Tel:** 03/5856202– 5856458 & **Fax:** 03/5828988 & **Mob:** 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com

Instruction For Use

ISSUE/REV: - 2/0

Page 2 of 2

ULTRAVAC SET