



Ultra for medical Products (ULTRAMED) U.M.I.C (S.A.E)

## INSTRUCTION FOR USE FOR URINE COLLECTION BAG LEG BAG

### Material Used:

- L.D.P.E. (Low Density Polyethylene)
- Master Batch (Colored).
- P.P. (Polypropylene)
- H.D.P.E. (High Density Polyethylene)
- P.V.C. (Poly Vinyl Chloride) non phthalate and DEHP free.
- P.V.C. (Poly Vinyl Chloride) sheets.

### Indications:

- Used to collect the urine of the patient from the bladder in case of urinary retention through Nelaton or Foley's catheter and to measure the volume of urine for daily checks.

### Contraindications:

- Can cause infection to the patient.

### Instruction for Use:

- Strap the bag to the leg of the patient.
- Remove the cover of inlet connector from the tube and connect the tube with the Foley's or Nelaton catheter.
- The measuring scale on the bag provides an approximate and indicative measurement of the volume of the urine.
- Drain the urine using outlet connector if the bag is filled more than half of the capacity.
- It must be used by skilled and trained personnel.

### Cautions:

- Discard after single use.
- Use only if the package is intact

### Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room Temperature.

EC REP

Obelis<sup>SA</sup>

53 Bd. Général Wahis  
B-1030 Brussels, Belgium  
Phone: 32.2.732.59.54  
Fax: 32.2.732.60.03  
E-mail: mail@obelis.net  
www.obelis.net

CE 2803



Sterilized by Ethylene Oxide



For Single Use Only



Cautions



Non-Pyrogenic



Don't use if package is not intact



Consult Instruction for Use

LOT

Lot No.



Date of Manufacturing



Use by / Expiry date

MD

Medical Device



Manufacturer Address



Latex Free

REF

Catalogue No.



NON-DEHP

Non-phthalate



ISSUE/REV DATE: 01.09.2022

### Instruction For Use

ISSUE/REV: - 2/0  
Page 1 of 1

URINE BAG LEG BAG