



ULTRA FOR MEDICAL PRODUCTS CO.

INSTRUCTION FOR USE FOR

ULTRAMED SURGICAL GLOVES (POWDER FREE)

Material Used:

- Natural Rubber Latex powdered with Corn Starch (Powder free).

Indications:

- To cover hands during Surgery.

Contraindications:

- Not to be used in patients with known hypersensitivity to any of the materials used.

Instruction for Use:

- Wash your hands before donning surgical gloves whenever possible. Pull a surgical glove out of the glove inner paper with one hand. Most people start by holding the glove in their non-dominant hand.
- Holding the glove by the open edge with the fingers of your non-dominant hand, make sure that the glove is facing the right direction to facilitate the thumb without fumbling.
- Slide your opposite hand into the base of the glove, fingers spread slightly apart. Make sure that you find each opening with the tips of your fingers before you push your hand all the way in. Wiggle your fingers slightly to make sure that the glove is seated firmly and comfortably.
- Pull the other glove out of the glove inner paper and repeat the process with the other hand.
- Examine the gloves to make sure that there are no holes or tears prior to contact with a patient or any surface.

Cautions:

- Single use only.
- Sterile until package is open or damaged.
- After donning remove powder by wiping glove thoroughly with a sterile wet sponge, sterile wet towel or other effective method.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room Temperature.



Obelis^{SA}

53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide



For Single Use Only



Cautions



Lot No.



Date of Manufacturing



Use by / Expiry date



Medical Device



Non-Pyrogenic



Catalogue No.



Manufacturer Address



Don't use if package is not intact



Latex



Consult Instruction for Use



Non-phthalate



ISSUE/REV DATE: 01.09.2022

Instruction For Use

ISSUE/REV: - 2/0

Page 1 of 1

ULTRAMED SURGICAL GLOVES (POWDER FREE)