

## **INSTRUCTION FOR USE FOR URINE COLLECTION BAG WITH SAMPLING PORT**

### **Material Used:**

- L.D.P.E. (Low Density Polyethylene)
- Master Batch (Colored).
- P.P. (Polypropylene)
- H.D.P.E. (High Density Polyethylene)
- P.V.C. (Poly Vinyl Chloride).
- P.V.C. (Poly Vinyl Chloride) sheets.
- A.B.S. (Acrylonitrile butadiene styrene)

### **Indications:**

- Used to collect the urine of the patient from the bladder in case of urinary retention through Nelaton or Foley's catheter and to measure the volume of urine for daily checks.

### **Contraindications:**

- Can cause infection to the patient.

### **Instruction for Use:**

- Remove the cover of inlet connector from the tube and connect the tube with the Foley's or Nelaton catheter.
- The measuring scale on the bag provides an approximate and indicative measurement of the volume of the urine.
- Drain the urine using outlet connector if the bag is filled more than half of the capacity.
- Use the sampling port for taking sample easily.
- It must be used by skilled and trained personnel.

### **Cautions:**

- Discard after single use.
- Use only if the package is intact

### **Conditions of Handling, Preservation and Storage:**

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room Temperature.



**Obelis<sup>SA</sup>**  
53 Bd. Général Wahis  
B-1030 Brussels, Belgium  
Phone: 32.2.732.59.54  
Fax: 32.2.732.60.03  
E-mail: mail@obelis.net  
[www.obelis.net](http://www.obelis.net)



Sterilized by Ethylene Oxide



Lot No.



For Single Use Only



Date of Manufacturing



Cautions



Use by / Expiry date



Medical Device



Non-Pyrogenic



Catalogue No.



Manufacturer Address



Don't use if package is not intact



NON-DEHP

Non-phthalate



Latex Free



Consult Instruction for Use

**ISSUE/REV DATE: 01.09.2022**

**Instruction For Use**

ISSUE/REV: - 2/0

Page 1 of 1

**URINE BAG WITH SAMPLING PORT**

