



ULTRA FOR MEDICAL PRODUCTS CO.

INSTRUCTION FOR USE FOR EXTENSION TUBE HIGH PRESSURE

Material Used:

- P.P. (Polypropylene)
- Medical grade phthalate free P.V.C. (Poly Vinyl Chloride) (DEHP free).
- Poly Carbonate.
- L.D.P.E. (Low Density Polyethylene).

Indications:

- Infusion of fluid such as parenteral nutrition and administration of other drugs and medicines.
- Maintaining hydration and/or correct dehydration in patients who are unable to take sufficient volume of oral fluids.
- Male luer lock at one end & female luer lock at other end.
- High Pressure Extension Line.
- For pressure up to 54 bar (800psi).

Contraindications:

- Administration of highly viscous fluids.
- Not to be used in patients with known hypersensitivity to any of the materials used.

Instruction for Use:

- Close completely flow control clamp of I.V. Infusion Set.
- Connect the Female luer connector of Extension Tube to the male fitting of Infusion set.
- Remove the protective cap of male lure connector and open roller clamp of I.V. Sets. Expel the air from the PVC tube and close roller clamp.
- Connect the male lure connector to the injection site (I.V. Cannula etc.) and regulate the flow by adjusting roller clamp to achieve desired flow rate.

Warning:

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

Cautions:

- Do not use if package is open or damaged.
- Discard after single use.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature.



ULTRA FOR MEDICAL PRODUCTS CO.



Obelis^{SA}

53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide



For Single Use Only



Cautions



Lot No.



Date of Manufacturing



Use by / Expiry date



Medical Device



Non-Pyrogenic



Catalogue No.



Manufacturer Address



Don't use if package is not intact



NON-DEHP

Non-phthalate



Latex Free



Consult Instruction for Use

ISSUE/REV DATE: 01.09.2022

Asheer



Assiut Factory: Part No. (304:310) & part no. (312) – Industrial Area, Arab El Awamer – Abnoub - Assiut – Egypt. Tel: 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax: 002-088-4964222 **& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaay Triumph Square Heliopolis Tel: 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

Assiut Office: 23, July Str. Tel: 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

Alexandria Office: 212 Abd El salaam Airef Str, Luran Tel: 03/5856202– 5856458 **& Fax:** 03/58289888 **Mob:** 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com

Instruction For Use

ISSUE/REV: - 2/0

Page 2 of 2

EXTENSION TUBE High Pressure