



ULTRA FOR MEDICAL PRODUCTS CO.

INSTRUCTION FOR USE ULTRAMED ULTRAWAY THREE WAY STOPCOCK

Material Used:

- Poly Carbonate.
- P.P. (Polypropylene)
- H.D.P.E. (High Density Polyethylene)
- Master Batch (White).
- Master Batch (Blue).

Indications:

- Infusion of fluid such as parenteral nutrition and administration of other drugs and medicines.
- Maintaining hydration and/or correct dehydration in patients who are unable to take sufficient volume of oral fluids.
- Transfusion of blood or blood components.

Contraindications:

- Administration of highly viscous fluids.
- Blood transfusion
- Not to be used in patients with known hypersensitivity to any of the materials used.

Instruction for Use:

- Three Way Stop Cock is connected into intravenous (I.V. Cannula) for delivery of two fluids or drugs at a time.
- Inspect Three Way Stop Cock package for integrity and expiry and then remove from package.
- Remove Luer Lock Plug of I.V. Cannula and insert male part of Three Way Stop Cock into female part of I.V. Cannula.
- Connect I.V. Infusion set to both female parts of Three Way Stop Cock after removing Luer Lock Plug.
- The Three Way Stop Cock can be used to administer two drugs or fluids at the same time.

Warning:

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

Cautions:

- Read Instruction before use.
- This product should be used by a qualified Doctor or a Paramedic.
- Check the product and package integrity prior to use and also the expiry date.
- The product is sterile, non-pyrogenic and non-toxic if package is not opened or damaged.
- For Single use only. Discard after single use. Do not recycle, clean or resterile.
- Use the product immediately after opening the individual blister packing.
- Store in a cool & dry place. Protect from excessive heat or direct sunlight.
- The product should be used according to instruction for use.

ULTRA FOR MEDICAL PRODUCTS CO. DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

Conditions of Handling, Preservation and Storage:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature.



ULTRA FOR MEDICAL PRODUCTS CO.



Obelis^{SA}

53 Bd. Général Wahis
B-1030 Brussels, Belgium
Phone: 32.2.732.59.54
Fax: 32.2.732.60.03
E-mail: mail@obelis.net
www.obelis.net



Sterilized by Ethylene Oxide



For Single Use Only



Cautions



Lot No.



Date of Manufacturing



Use by / Expiry date



Medical Device



Non-Pyrogenic



Catalogue No.



Manufacturer Address



Don't use if package is not intact



Non-phthalate



Latex Free



Consult Instruction for Use

ISSUE/REV DATE: 01.09.2022



Assiut Factory: Part No. (304:310) & part no. (312) – Industrial Area, Arab El Awamer – Abnoub - Assiut – Egypt. Tel: 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax: 002-088-4964222 **& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis Tel: 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

Assuit Office: 23, July Str. Tel: 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

Alexandria Office: 212Abd El salaam Aired Str, Luran Tel: 03/5856202– 5856458 **& Fax:** 03/5828988 **& Mob:** 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com